# PLUSCUR® Flexi Depth

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Art.-Nr.: P206080

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## Flexible wound depth measurement

Item no.: P206080

Sterile product.  
Sterilised with ethylene oxide.  
Disposable product.

Check the outer packaging for sterility when opening the product. Do not use product if the packaging is damaged or opened.  
Storage conditions: Store in a dry place at (<25 °C).

### DESCRIPTION:

PLUSCUR® Flexi Depth gauge is a flexible measuring stick made from medical PVC that can be manually adjusted to the wound. The rounded tip enables an atraumatic measurement inside the wound bed, also in the presence of fistulas and wound pockets. The PLUSCUR® Flexi Depth gauge has a depth scale of 15 cm divided into 1 mm increments. The numbers from 1-15 cm correspond to the wound in cm.

### PURPOSE:

PLUSCUR® Flexi Depth gauge is a gauge used by medical professionals to measure the depth of a wound.

Its unique flexibility allows the user to measure the deepest part of the wound; fistulas or wound pockets can also be easily measured. After inserting the PLUSCUR® Flexi Depth gauge all the way to the wound bed, the depth of the wound can be read on the measuring scale. Within the scope of continuous documentation, the measurements serve to measure and monitor wound healing from the depth.

In addition to measuring the wound depth, the width, length and orientation of the wound on the respective part of the body should always be recorded as well. Please use a wound ruler, as described, for correctly measuring the dimensions of the wound.

### INDICATION:

PLUSCUR® Flexi Depth gauge can be used to measure the depth of all kinds of wounds.

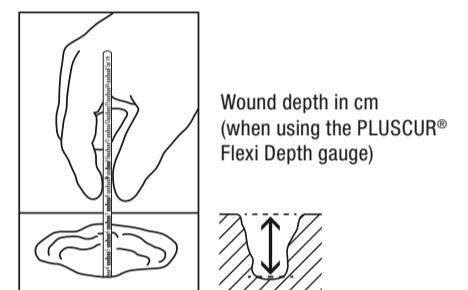
### INSTRUCTIONS FOR USE:

1. Cleaning: Make sure the wound is properly and duly cleaned. After that, carefully tear open the sterile packaging at the upper end (15 cm) and remove PLUSCUR® Flexi Depth gauge. Make sure to do this only under sterile conditions.
2. Measuring: Insert the PLUSCUR® Flexi Depth gauge with light pressure into the wound perpendicular to the wound bed.
3. Marking: Mark the measured depth on the PLUSCUR® Flexi Depth gauge / skin level.

4. Disposal: Dispose of the PLUSCUR® Flexi Depth gauge correctly.

### PRECAUTIONS:

- PLUSCUR® Flexi Depth gauge is designed for single use only and must not be reprocessed. There is a risk of cross-contamination.
- The rounded tip is meant to be inserted into the wound; please do not insert the other end of the PLUSCUR® Flexi Depth gauge into the wound.
- Insert the rounded tip into the wound with lightest pressure and rest it on the wound bed.

Product subject to changes by the manufacturer.

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BMV® Bender Medical Vertrieb GmbH · www.bmv-med.de